DE

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EN ISO 374-1:2016 Protective gloves against dangerous chemicals and micro-organisms

Table with 5 columns: ISO 374-1:2016/ Type B, Test chemical, Code letter, Class, Test result. Includes rows for n-heptane, Sodium hydroxide 40%, and Formaldehyde 37%.

Table with 2 columns: Test chemical, Degradation (%). Includes rows for n-heptane, Sodium hydroxide 40%, and Formaldehyde 37%.

EN ISO 374-5:2016 Protective gloves against dangerous chemicals and micro-organisms

Table with 2 columns: ISO 374-5:2016, VIRUS.

This information does not indicate the actual duration of protection at the workplace and the distinction between mixtures and pure chemicals. The resistance to chemicals has been assessed under laboratory conditions on samples taken only from the palm of the hand (except where the glove is 400 mm or longer - in which case the cuff is also tested) and refers exclusively to the chemicals tested.

The decontamination of chemical and biological contamination must be carried out specifically. The contamination must be known both qualitatively and quantitatively in order to be able to make a statement about the degree of decontamination. In any type of decontamination, self-protection is important to avoid an endangerment of the person and the environment.

These gloves protect against microorganisms (bacteria and fungi), viruses. The resistance to penetration was assessed under laboratory conditions and relates exclusively to the samples tested.

Icons and text for food contact, AQL < 1,5, marking medical device, Manufacturer, Manufacture date, Expiry date, Not reusable, EAC marking, UkrSepro marking, Read the manufacturer's instructions and information, CE marking.

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EN

Manufacturer's instructions and information

Information brochure for personal protective equipment (PPE) according to Regulation (EU) 2016/425, annex II point 1.4. Please read this information brochure carefully before using the PPE.

Table with 2 columns: Protective gloves, Risk category III. Includes Size(s), Certification, and Notified body.

The CE marking certifies that the product complies with the essential health and safety requirements of Regulation (EU) 2016/425. The EU declaration of conformity can be viewed at www.doc.nitras.de.

This product is personal protective equipment of risk category III. It protects you against risks that can lead to very serious consequences such as death or irreversible damage to health.

Storage / use / servicing: Store in a cool, dry place. Keep away from direct sunlight, UV rays or ozone sources. Do not store in buckled condition or under weight load. If possible, store or transport the product in its original packaging.

Instructions for wearing the product: Make sure your hands are clean and dry before putting on gloves. Insert your fingers into the respective glove and pull the glove loosely over your hand on the knitted wrist or cuff.

A suitable skin protection product can be used before starting work (after breaks and if necessary after washing the hands). During work (before breaks and before end of work) a suitable skin cleanser can be used.

Disposal: Dispose of with household waste. This product may be contaminated by environmentally harmful or hazardous substances after intended or unintended contact with chemicals.

General explanations of achieved performance levels: 1-6 Achieved test result (the higher, the better). 0 Minimum performance level not achieved. X Not tested or not applicable due to the material or design.

EN 420:2003 + A1:2009 Protective gloves - General requirements and test methods

Table with 3 columns: Test parameter, Performance level, Test result. Includes Dexterity.

This article is intended to protect the user's hand from substances, liquids or chemicals for a short period of time. Therefore, these are gloves for special applications and the length of the gloves deviates from the dimensions according to EN 420.

8325 // MINT WAVE Schutzhandschuhe / Risikokategorie III Protective gloves / Risk category III

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